

Prostate Examination Simulator

Instruction Manual


This simulator is designed to facilitate prostate examination training.

Do not use it for improper purpose. Please read the instruction carefully before use.

■Features

- 3 patient positions can be simulated: supine, prone and lateral position.
- Ten different prostate models including normal prostate are prepared for training.
- Prostate models can be changed over smoothly by rotating the dial.

■Set includes

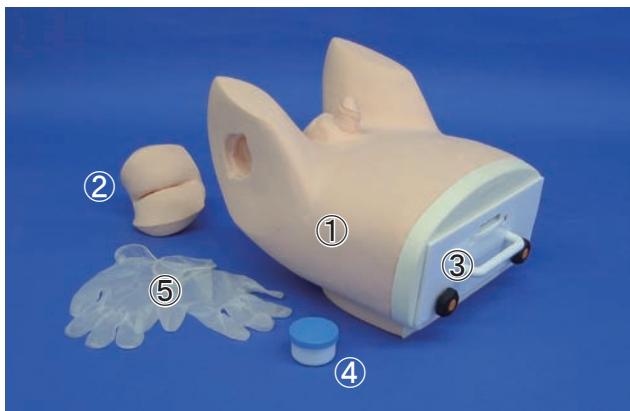


- ① Lumbar torso
- ② Anus pad
- ③ Rotation unit: 10 prostate models are installed
 - Normal: 2 variations
 - Enlargement: 2 stages
 - Carcinoma: 5 variations
 - Prostatitis: 1
- ④ Vaseline
- ⑤ Gloves
- Storage case (no image)

■Training instructions


Select a case by rotating the dial.

○Place the torso model as appropriate patient position.

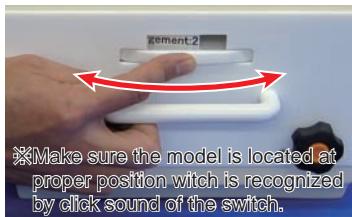

■Replacement of the model parts

○Replacement of prostate models

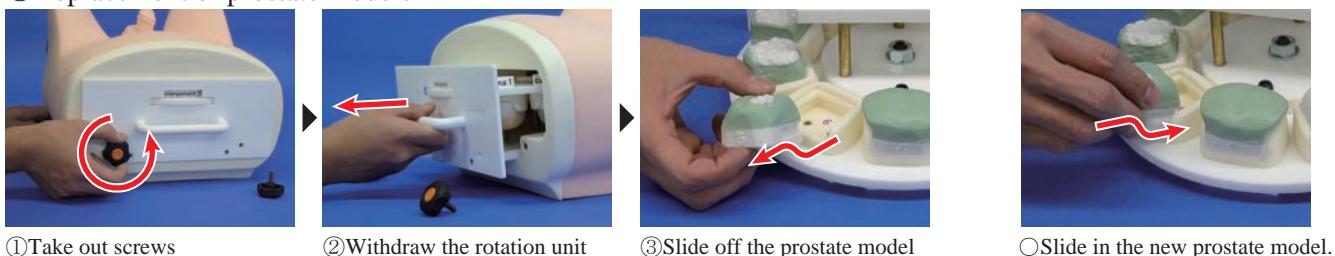

①Take out screws

②Withdraw the rotation unit

③Slide off the prostate model which is to be replaced.

○Slide in the new prostate model.

○Replacement of anus pad



Dismount:
Gently, raise the edge around the unit and pull the unit straight off from the lumbar model.

Insert a new unit into the opening and fit the edge.

DOs and DON'Ts

DOs

- The materials for the models are special compositions of soft resin.
- Please handle them with the utmost care at all times
- Please coat your finger(s) with vaseline, when you perform the training.
- Store the training set at room temperature, away from heat, moisture and direct sunlight.

DON'Ts

- Never wipe the simulator pads with thinner or organic solvent.
- Don't mark on the phantom and models with pen or leave any printed materials in contact with their surface. Ink marks on the models won't be removable.